

KAYASAFETY.COM

DATA SHEET

RL-SLING
FOUR POINT BRIDLE FOR
SLING STRETCHERS



Failures and Repairs

The possible causes and the solution are mentioned below in case of any failures detected regarding the product. The customer service must be consulted if a failure is detected other than the faults specified here.

Problem : Webbing or carabiner is not opening/not closing.

Possible Cause: Lock may have been broken. **Solution:** Product must be replaced with a new one. **Problem:** There is a fracture or crack on the surface.

Possible Cause: Overloading.

Solution: Put the sling out of service.

Warning: Replacement of original parts with

parts of another manufacturer can affect product functions and terminate $% \left(1\right) =\left(1\right) \left(1\right) \left$

the guarantee.

RL-SLING

FOUR POINT BRIDLE FOR SLING STRETCHERS

(EN)

- Made of 45 mm high tenacity polyester webbing
- Adjustable legs: 160 cm (Max. Length 240 cm)
- Carrying rings (10 kN)
- 4 x K-2 model karabiner (23 kN, EN 362)
- High tenacity bounded type UV and dust resistant sewing yarn.
- Suitable to use with RL-4 and RL-4A model stretchers.
- Can be used with basket stretchers by attaching from 4 points

Control

Must be checked against any damage anyhow, and, before use, webbing and carabiners must be checked whether they are functioning properly.

The product should be put out of service in case any damage is detected, and the customer service must be informed.

Cleaning

Should be cleaned with a wet soapy sponge and dried thoroughly. Abrasive cleaning agents such as bleach or ammonia should never be used.

Scratching tools such as brushes, wire brushes, knives, etc. should not be used to remove stains.

Storage

Should be stored in a safe, secure and dry place in its packaging if it is not used immediately.